

I N S T R U C T I O N S



Fishing Pole Holder For Wheelchairs

70663-1000

1. Attach the clamp around the tubing (for diameters smaller than 1" (2.54 cm) use spacer provided)
2. Insert the bolt through the large hole and into the small hole of the plastic tube
3. Push the bolt through the clamp holder and attach the wing nut onto the bolt end
4. Position the tube so the slotted end is tilted upward and outward, away from the front of the chair
5. Tighten the wing nut and insert the pole so the reel stem slides into the slot of the tube



Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com