

I N S T R U C T I O N S



Bowling Ball Pusher

Catalog Number 71205-0000

To Assemble:

Slide bowling ball pusher assembly into the handle assembly through the anti-rattle cap. Twist the handle so that the internal cam will twist and lock on the inner diameter of the handle tube. The total length of the device may be adjusted as required.

To Use:

Place bowling ball on the floor. Slide the pusher up to the ball until the bumpers on the pusher meet the ball. The bowler may now push the bowling ball down the alley as the bumpers on the bottom of the pusher slide along the floor.



NOTE:

The pusher must make good contact with the ball before pushing off and must slide on the floor for stability. The bowling ball pusher may be used from a wheelchair or from a standing position.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com