

# I N S T R U C T I O N S



## Tall-Ette® Elevated Toilet Seats With Slip-In Lok-In-EI® Bracket

FOR CATALOG NUMBERS: 72569-1000, 72580-2000,  
72581-2000, 72584-1000, 72585-1000, 72586-1000,  
72590-1000, 72592-1000, 72595-0002, 72597-1000

Place the Tall-Ette® Elevated Toilet Seat upside down on a table or bench with its two bracket holes visible. Hold the Lok-In-EI® Bracket with the two pegs lined up with the holes on the seat. The bracket tongue should be facing away from the seat. Press the pegs into the holes until the bracket is in place.

### To Install on Toilet:

1. Lift the toilet seat to a raised position.
2. Hold the Tall-Ette® Elevated Toilet Seat so that the Lok-In-EI® bracket tongue points at a downward angle and slide the Lok-In-EI® bracket under the hinge pin between the bolts that hold the toilet seat in place.
3. As the tongue slides into place, the Tall-Ette® Elevated Toilet Seat should come to rest directly on the china bowl with the bottom flange fitting inside the rim of the toilet bowl.
4. Now sit on the Tall-Ette® Elevated Toilet Seat. The weight of your body will tighten the Lok-In-EI® bracket's peg joint if it had not previously been seated completely.

### To Clean:

Wash with a mild soap and water solution. **DO NOT USE** commercial cleaners or bleaches, they may discolor the seat. If an antiseptic is desired, apply plain alcohol and wipe off promptly.

### Caution:

**Do not use the Tall-Ette® Elevated Toilet Seat if it does not fit into the toilet bowl rim.**

If you require further assistance in the installation, operation or use of this product please contact us.

## Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)

972588606

1206

# I N S T R U C T I O N S



## Tall-Ette® Elevated Toilet Seats Without Slip-in Lok-in-EI® Bracket

### To Use:

Lift the toilet bowl to a raised position and place the Tall-Ette® Elevated Toilet Seat directly on the porcelain bowl. After use and removal of the Tall-Ette® Elevated Toilet Seat, lower the toilet bowl seat to its normal position for safety.

### To Clean:

Wash with a mild soap and water solution. **DO NOT USE** commercial cleaners or bleaches, they may discolor the seat. If an antiseptic is desired, apply plain alcohol and wipe off promptly.

### Caution:

**Do not use the Tall-Ette® Elevated Toilet Seat if it does not fit into the toilet bowl rim.**

If you require further assistance in the installation or use of this product please contact us.

## Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)