


I N S T R U C T I O N S



Roll Easy™ Lotion Applicator

Catalog Number 74133-0050

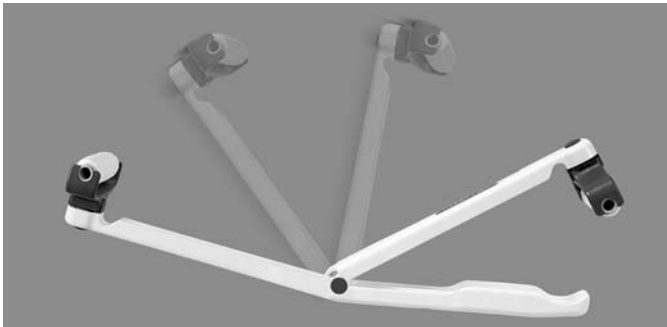

Massages the body while applying ointments, creams, lotions and medications. The inverted roller is perfect for applying lotions to the arms, legs and neck. The round roller is ideal for use on the back.

Inserting the roller:

1. Unfold the Roll Easy™ Lotion Applicator. Be sure the handle is locked in place before using.
2. Snap the ends of the roller into the notches located on the head of the applicator.
3. Make sure that both ends of the roller are firmly snapped into the head of the applicator.

Changing the roller:

1. Gently pull the roller out of the head of the applicator.
2. Simply snap the other roller into place.
3. Make sure that both ends of the roller are firmly snapped into the head of the applicator.

Applying lotions with the Roll Easy™:

1. Insert desired roller into the head of the applicator.
2. Place lotion on the roller.
3. Gently glide roller along the body.
4. When finished simply wash the roller with water and a mild soap.

Changing the angle of the Roll Easy™ Lotion Applicator:

The head of the Roll Easy™ pivots allowing the rollers to be used at any angle. Gently turn the head of the roller in either a clockwise or counterclockwise direction until the desired angle is obtained.

Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com