

I N S T R U C T I O N S



People Feeder

Catalog Number: 74581-0001

Bag of 2 Replacement Nozzles: 74581-0010

This product is to be used only with soups or pureed food.


Eating Instructions:

1. Pinch rubber nozzle with thumb and index finger.
2. Push the plunger down and move the food through the nozzle by squeezing the nozzle (refer to diagram below).
3. Continue to push the plunger and move the food through the nozzle until the nozzle is full. The nozzle holds one teaspoon of food.
4. Feed the person by squeezing the food out of the nozzle into the mouth.




Caution: Feeding by only pushing the plunger will result in difficulty using this product.

Caution: Make sure patient does not bite nozzle.

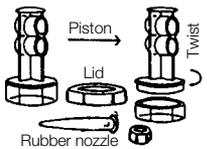

All parts can be detached.
Rubber nozzle is marked with the letter L.

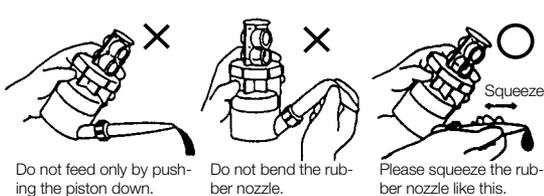

Do not feed only by pushing the piston down.

Do not bend the rubber nozzle.

Please squeeze the rubber nozzle like this.

Cleaning Instructions:

1. To thoroughly clean the nozzle, rub it in your hands while running water through it.
2. When boiling or sterilizing please separate all parts.
3. When sterilizing, please use temperatures lower than 248° F (120° C) for no more than 10 minutes.



Do not use a brush.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com