

# I N S T R U C T I O N S



## One-Hand Cutlery Set

CATALOG NUMBER 74610-0000



This fork/knife combination makes one-handed eating easy! When pushed down onto food, the fork retracts inside the handle as the knife cuts a bite-sized piece of food. When pressure is released, the fork emerges from the knife with the food. A raised lip guard protects the lips from the knife. Dishwasher safe.

### To Use:

Place the fork on food and press down. Rock the fork back and forth a few times. After food has been cut, release pressure on the utensil so that the food filled fork emerges from the knife.



## Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)