

I N S T R U C T I O N S



Bed Rope Ladder

Catalog Number 76447-0000



CAUTION: This ladder is not designed for climbing.

1. The rope ladder is made of a single strand of rope. If your bed has a cross rail, tie the end loop of rope around it.
2. If your bed is of a different design, cut the rope at the center of the loop and tie each end to an available rail.
3. If it is necessary to tie the rope to the front legs or bedposts, cut the rope loop at the center and knot each end. Then tie on the required length of extra rope (such as clothesline) so that it can be tied to the legs or bedposts.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com