I N S T R U C T I O N S

Single

Model 76488-0000

Double

Model 76488-0010

Single AbleRise™ Bed Assist

Model 76488-0000

Parts:

- A. Handle
- B. Angle Bracket (2 pieces)
- C. Extension
- D. 1½" Screw (2 pieces)
- E. Nut (2 pieces)

Assembly Steps:

- 1.) Assemble two angle brackets (B) to handle (A) as shown in Fig.1 using 1½" screws (D) and nuts (E) provided.
- 2.) Insert ends of angle brackets (B) into the ends of extension (C) as in Fig.1 and engage push-pins with the holes on extension (C).
- 3.) Insert extension (C) fully between mattress and box-spring until handle (A) butts against the edge of the mattress.
- 4.) Secure extension (C) to bed frame using attached strap. For beds requiring a longer strap than provided, use any sturdy cord or strap commercially available.

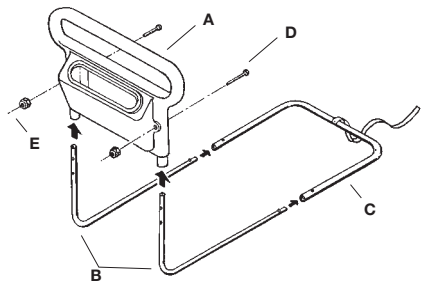
Fig. 1

Double AbleRise™ Bed Assist

Model 76488-0010

Parts:

- A. Handle
- B. Angle Bracket (4 pieces)
- D. 1½" Screws (4 pieces)
- E. Nut (4 pieces)
- F. Connector (2 pieces)

Assembly Steps:

- 1.) Assemble brackets (B) to two handles (A) by following Step 1 for the Single AbleRise™ as shown in Fig. 1.
- 2.) Insert the ends of brackets (B) into two connectors (F) as shown in Fig. 2 and engage push-pins with holes on the connectors (F) as follows:
 - (a) 1st holes from the ends for a "Queen" size bed.
 - (b) 2nd holes from the ends for a "Full" size bed.
 - (c) 3rd holes from the ends for a "Twin" bed.
- 3.) Place mattress over connectors (F). Make sure the handles butt against the mattress.

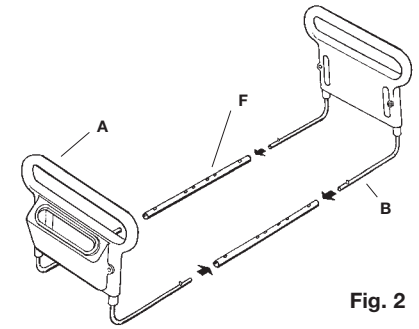
Fig. 2

Warning:

- 1.) The AbleRise™ Bed Assists are designed to be used only as an aid for getting on and off the bed. Do not lean, pull or push against the handles with full body weight.
- 2.) Frequently check all screws, push-pins and straps and reposition or tighten when needed.
- 3.) Loose straps can be dangerous to children. Make sure to keep children away from the straps to prevent choking hazards.

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Maddak Inc.