

I N S T R U C T I O N S



Tuffet™ Leg and Foot Rest

Catalog Number 76630-0000

The Tuffet™ Leg Rest provides comfort in two different modes. With the “V” side up, it is a foot and leg rest that rocks to the most comfortable position for you. Just put your heels into the curve with your legs against the long side and rock it to your satisfaction.

Flip it over and the broad surface creates a comfortable rest to put your feet on.

Although firm, it has a soft feel. The firmness can be increased by blowing air into the valve.

To Release Air: Use something similar in diameter to a pencil point to release the air. GENTLY place the point into the valve and remove it when enough air has been released.

Clean with a damp cloth and soap or detergent. Wipe with a wet cloth to rinse off.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com