

# I N S T R U C T I O N S



## Wheelchair Laptop Desk

Catalog Number 70601-0000

- 1) Lay the arms of the wheelchair laptop desk on the arms of the wheelchair.
- 2) Wrap the strips of hook and loop fastener around the arms and secure tightly. Refer to Figure 1.

If used on a wheelchair that has removable arms the desk does not have to be removed from the arms before removing the arms from the wheelchair.

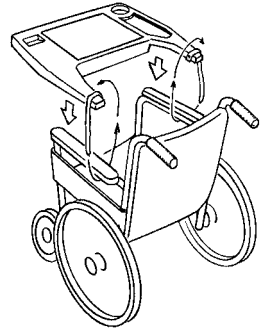


Fig. 1

The desk can be used in several different ways:

For a flat writing surface place the tray with the smooth side up and the edge with the rope closest to you.

For an inclined surface place the tray the opposite way.

To use the book/paper holder place the tray through the slots on the sides of the desk. Refer to Figure 2. The book holder folds and unfolds by rotating the sides about the center axis.

Remove the lid from the upper right hand corner of the desk in order to use the drink holder.

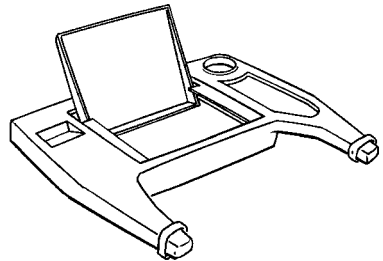


Fig. 2

**WARNING:** Do not put more than ten pounds in the storage part of the desk.

## Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)