

I N S T R U C T I O N S



Bowling Ramp

Catalog Numbers: 71206-1000, 71206-2000, 71206-2001

To Assemble :

With the bottom of the ramp on the floor, slide the top of the ramp over the pins extending out from the side of the "U" frame. The ramp must slide over the pins completely to meet the "U" frame for stability. When the ramp is in place, the push pin should pop through the hole to lock in place.

To Use :

The bowling ramp should be placed in front of the lane, facing down the alley. An assistant places the bowling ball on top of the frame. The bowler lines up a direct path to the bowling pins and gives the ball a slight push. The ball will then roll down the ramp, down the alley and towards the pins. The ramp may be used by an individual standing or in a wheelchair.



To Disassemble:

Hold the "U" frame, push pin inward, and slide the rails off.

Caution:

If the "U" frame is not held during disassembly, it may fall over and cause injury.

To Clean:

Clean with damp cloth and detergent. Do not use abrasive cleaners.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com