

# I N S T R U C T I O N S


## Lock-On™ Elevated Toilet Seat

**Note:** Inspect all components for shipping damage. If any parts are missing or damaged, contact your dealer/retailer for replacements.

### To Install:

Note - **DO NOT** use tools. Hand-tighten only. **DO NOT** over tighten knob. Check seat for proper installation and realign/ retighten if necessary.

1. The existing toilet seat and lid can either be removed or flipped up depending on user preference.
2. Turn the knob (marked A) counterclockwise and completely loosen the clamp (marked B) on the front of the elevated seat. This will place the clamp in its longest adjustment.
3. Position the rear wedge (marked C) under the rear rim of the toilet bowl. Note: The elevated seat **MUST** be inside the toilet bowl. The rear wedge **MUST** be positioned under the rear lip of the toilet bowl.
4. After the elevated seat is properly placed into the toilet and the clamp is engaged with the front of the bowl, turn the knob clockwise until the seat is securely fastened.

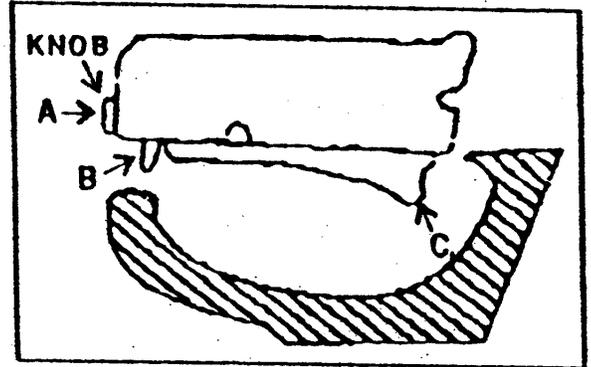

### To Remove Elevated Toilet Seat:

1. Rotate the knob counterclockwise until the elevated seat is loose.
2. Lift up the front of the elevated toilet seat until the adjustment clamp clears the lip of the toilet bowl.
3. Pull the seat towards you at an angle to remove the elevated seat from the toilet bowl.

### Care and Maintenance:

Do not use an abrasive cleaner. Use a non-abrasive cleaner with warm water.

Periodically inspect all components for damage or looseness. **DO NOT USE IF THE SEAT IS LOOSE!** Retighten using enclosed directions.

### Warning:

- Do not use this product without first reading and understanding the instruction sheet. Otherwise, injury or damage may occur.
- Elevated seats with arms are not designed to support total body weight. They should be used for assistance only.
- Weight capacity - 300 lb. (136.36 kg)