

I N S T R U C T I O N S



Bedside Beverage Holder

CATALOG NUMBER 74576-0002

1. Open the clamp and place it around frame of the bed or wheelchair.
2. Slide the threaded rod at the end of the gooseneck through the clamp.
3. Screw the lever knob onto the rod making sure the cupholder is in an upright position.
4. Tighten the lever knob.

For frames under 1" in diameter, a plastic reducer is supplied. Position the reducer on the frame and put the clamp over it.

Caution: Hot beverages should not be placed in the beverage holder.

Caution: A container holding more than 20 oz. of fluid should not be placed in the beverage holder.



Maddak Inc.

6 INDUSTRIAL ROAD, PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com