

I N S T R U C T I O N S



Shampoo Rinse Tray

Catalog Number 76428-0001

Gather all necessary equipment:

- A. Shampoo Rinse Tray
- B. 2 or 3 pitchers of warm water
- C. Shampoo and conditioner
- D. Towels
- E. Pail



Instructions:

1. Position the head so it is close to the side of the bed.
2. Slide the pillow down so that it supports the shoulders and neck.
3. Place the Shampoo Rinse Tray under the head so that the neck lies comfortably in the neck rest.
4. Place the end of the drain hose into a pail on the floor. Do not pinch or compress the hose while shampooing—the hose should hang freely at the side of the bed to allow free flow of water into the pail.
5. Use the pitchers of water to wet hair. Proceed to shampoo as usual. Be sure to rinse hair thoroughly to avoid leaving soap residue in the hair.
6. If soaking the hair or scalp is desired, the hose may be closed by folding it in the middle to form a kink. Then slide the plastic ring over the end of the kink to hold it in place.
7. The Shampoo Rinse Tray should be wiped dry before storage.

For a **free** copy of the latest Maddak Catalog featuring more fine Ableware® products, please write to:

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com