

# I N S T R U C T I O N S



## Shoulder-Mounted Shampoo Rinse Tray

Catalog Number 76429-0000

### CONTENTS

- 1 White Tray
- 2 Foam Pads
- 2 Black Flaps

1. Insert the black flaps into the slots in tray from the top. When fully inserted, they will snap in place permanently.
2. Peel the protective film from the foam pad to expose the pressure-sensitive adhesive.
3. Press the adhesive side of the foam pad against the flap on the side that will contact the shoulder.
4. Do the same with the second pad and flap.



### INSTRUCTIONS FOR USE

1. Place a chair or wheelchair with its back toward a sink, so that the person in the chair faces away from the sink.
2. Place the tray on the person's shoulders, allowing the two flaps to hang in front of the shoulders.
3. The tray must slope downward toward the sink to drain into the sink. If the person is short, it may be necessary for him or her to sit on a pillow or riser to gain the needed slope.
4. The person can now tilt her/his head back into the curved neck rest so the hair is over the tray. Shampooing and rinsing can now commence, allowing drained water to flow into the sink.

# Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA  
973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)