


I N S T R U C T I O N S

Hinged Elevated Toilet Seat

Catalog Number 72571 Series

1. Remove existing seat and lid from the toilet
2. Place the two rubber washers (C) over the two bolt holes of the toilet
3. Place Hinged Toilet Seat Unit (A & B) on toilet, small slotted holes up, making sure the holes line up with the holes in the rubber washers (C)
4. Place existing toilet seat over the hinged toilet seat
5. Insert the bolt (D) through the toilet seat hinge, the mounting block of the hinged seat (B), the rubber washer (C) and the toilet
6. Repeat with the second bolt (D) in the other hole
7. Slip washer (E) onto the end of the bolt (D) and tighten nut (F). Repeat with second bolt

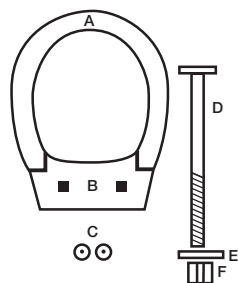

Parts
 A. Lifting Riser
 B. Mounting Block
 C. Rubber Washers
 D. Seat Bolt
 E. Washer
 F. Nut

**WARNING:**

HINGED TOILET SEAT SHOULD NEVER BE USED WITHOUT THE NORMAL TOILET SEAT ATTACHED AND HINGED IN THE DOWN POSITION




A Division of SP Industries
 661 Route 23 South • Wayne, NJ 07470
 973-628-7600 • email custservice@maddak.com • www.spableware.com

I N S T R U C T I O N S

Hinged Elevated Toilet Seat

Catalog Number 72571 Series

1. Remove existing seat and lid from the toilet
2. Place the two rubber washers (C) over the two bolt holes of the toilet
3. Place Hinged Toilet Seat Unit (A & B) on toilet, small slotted holes up, making sure the holes line up with the holes in the rubber washers (C)
4. Place existing toilet seat over the hinged toilet seat
5. Insert the bolt (D) through the toilet seat hinge, the mounting block of the hinged seat (B), the rubber washer (C) and the toilet
6. Repeat with the second bolt (D) in the other hole
7. Slip washer (E) onto the end of the bolt (D) and tighten nut (F). Repeat with second bolt

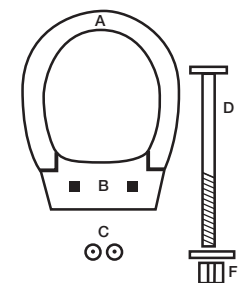

Parts
 A. Lifting Riser
 B. Mounting Block
 C. Rubber Washers
 D. Seat Bolt
 E. Washer
 F. Nut

**WARNING:**

HINGED TOILET SEAT SHOULD NEVER BE USED WITHOUT THE NORMAL TOILET SEAT ATTACHED AND HINGED IN THE DOWN POSITION



A Division of SP Industries
 661 Route 23 South • Wayne, NJ 07470
 973-628-7600 • email custservice@maddak.com • www.spableware.com