

I N S T R U C T I O N S

Shower Stool

72715-2000 Non-Rotating Shower Stool



72715-2100 Rotating Shower Stool

Parts List:

1. Seat - 1 each
2. Leg - 4 each
3. Turntable – 1 each (Rotating model only)
4. Slip disk – 1 each (Rotating model only)

To Assemble:

1. Place seat upside down on a flat surface
2. Insert legs into the recesses of the seat (see figure 1) by applying a firm downward pressure until the leg is fully engaged
3. Upright seat
4. Place slip disk on top of seat; aligning the center holes (rotating version only - see figure 2)
5. Place turntable over seat and insert the center stem through both center holes of the slip disk and seat (rotating version only - see figure 2)

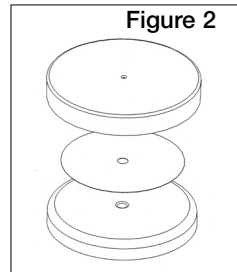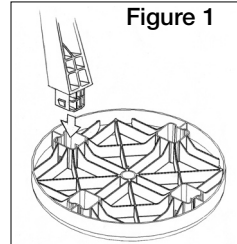

Adjusting the Legs:

1. Rotate the leg extension and disengage from the leg
2. Insert the leg extension into the desired slot until fully engaged

Care and Maintenance:

1. Clean with a non-abrasive cleaner
2. Periodically inspect all components for damage or looseness

Warning:

- Do not stand on the shower stool for any reason
- Do not use this product without first reading and understanding the instruction sheet
- Weight capacity - 300 lb. (136.36 kg)



A Division of SP Industries

661 Route 23 South • Wayne, NJ 07470

973-628-7600 • email custservice@maddak.com • www.spableware.com