



Novo Cup

Spill-free drinking even while lying down

- Eliminates the need to sit up or raise the head while drinking
- Spill-proof even when tipped over
- Can be used with hot and cold fluids
- Holds 8.5 oz. (250 ml)
- Engraved measurement markings in 25 ml increments
- Dishwasher, microwave and autoclave safe
- Comes with 11 straws
- Replacement straws available (745900001 pack of 10)



Item 745900000

Made in Denmark


SP Ableware
A Division of SP Industries
www.spableware.com




Instructions:




1. Fill the Novo Cup with liquid.

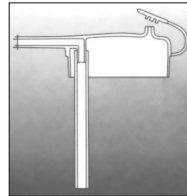

2. Fit the straw securely onto the short tube on the underside of the lid.

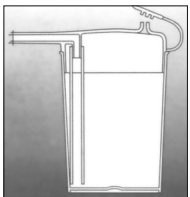

3. Make sure the valve on the lid is open and then place the lid on the cup. If the valve is closed liquid will spurt out of the cup. Press firmly down on the lid.

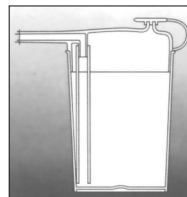

4. Close the valve on the top of the lid. After the first sip the cup will be spill proof.

 **Maddak**
SP ABLEWARE
www.spableware.com